

# Go Anywhere, Do Anything



WHILL fits in most vans and some SUVs and hatchbacks.

Fold down the rear seats and you can drive

WHILL into the cargo space using portable ramp.

Here is a list of vehicles we actually inspected.



## Suitable Vehicles for WHILL

Type	Manufacturer	Model	Suitability	Back seat of the car must be folded down	Back support cushion of Model A must be removed.	Note
Compact SUV	Honda	CRV	Good	○	○	
Compact SUV	Honda	HRV	Good	○	○	
Crossover	Prius	V	Bad	×	×	
Hatchback	Chrysler	PT Cruiser	Good	○	×	
Minivan	Mazda	5	Good	○	○	
Minivan	Toyota	Seinna	Good	○	×	
Sedan	Nissan	Leaf	Bad	×	×	
Sedan	Tesla	Model S	Bad	×	×	
SUV	Audi	Q7	Bad	×	×	
SUV	BMW	X5	Bad	×	×	
SUV	Cadillac	Escalade	Good	○	×	
SUV	Honda	Element	Good	○*	×	* Back seat of the car must be swiveled to the side..
SUV	Honda	Pilot	Good	×	×	
SUV	Infinity	QX60	Bad	×	×	
SUV	Infinity	QX80	Good	○	○	
SUV	Land Rover	Land Rover	Good *	○	×	* The tailgate has a unique shape which makes it a little hard to drive up the ramp. You need 2 people to get Model A into this car.
SUV	Lexus	RX	Bad	×	×	
SUV	Lexus	GX	Good *	○	○	* The bottom of the device gets caught on the ramp where the ramp meets the cargo floor. You need 2 people to get Model A into this car.
SUV	Mazda	CX-5	Bad	×	×	
SUV	Mercedes-Benz	M-Class	Good	×	×	
SUV	Mercedes-Benz	GL-Class	Good	×	×	
SUV	Nissan	Pathfinder	Bad	×	×	
SUV	Nissan	Rogue	Bad	×	×	
SUV	Range Rover	Range Rover	Good *	○	×	* The bottom of the device gets caught on the ramp where the ramp meets the cargo floor. You need 2 people to get Model A into this car.
SUV	Toyota	Highlander	Bad	×	○	
SUV	Toyota	FJ Cruiser	Good	○*	×	* Back seat of the car must be removed.
SUV	VW	Tiguan	Bad	×	×	
SUV	Toyota	RAV4	Good	×	○	
Wagon	Chevrolet	HHR	Good *	○	○	* WHILL's arms must be set to shortest length possible. Arm configuration requires the use of tools.
Wagons	Subaru	Outback	Bad	×	×	
Wagons	Toyota	Preus	Bad	×	×	