



WHILL Model A: Airline Handling Guide

For proper airport personnel handling of WHILL Model A personal mobility device.
Attach this document to device in a conspicuous area.

Owner's Contact Information:

Name: _____

Phone: _____

Device Serial #: _____



Batteries (2x)

Type: Gel/Dry Cell / Sealed Lead-Acid, Non-Spillable (AGM)
Model: Long Battery WP50-12NE / 12V 50Ah
Other: Securely attached to device. Does not require removal for airline travel



At destination, safely return the device to the owner at the gate by following steps 1 and 2 in reverse order.

WHILL Model A Personal EV meets the following guidelines for safe airline transportation: Federal Aviation Administration (FAA); Transportation Security Administration (TSA); Department of Transportation (DOT); International Air Transport Association (IATA). However, the device has not been specifically tested for airline transportation.