




WHILL Model A: User Handling Guide for Air Travel

For proper user handling and preparation of WHILL Model A personal mobility device during airline travel.
Keep this document on hand for quick reference.

Flight Departure: Handing Off to Airport Personnel


1



Power Down

Turn Model A off, then completely rotate both arms up.


2



Document

Take pictures of the device in case damage occurs during transport. Wrap arms in bubble wrap to help prevent scratches (optional).

3

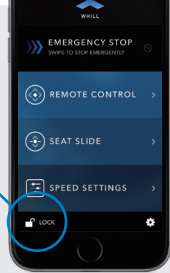


Lock Device

Use the Model A iPhone app to lock the device, or insert an "Air-Safe" plug (sold separately) into charger port

Flight Arrival: Receiving and Driving


1



Inspect

Inspect your Model A upon receipt. Document any damage that occurred during transportation.


2



Engage Brakes

Turn the two brake levers up to re-engage the brakes (located below the seat). Remove "Air-Safe" plug or unlock device using iPhone app.

3



Drive

Sit in Model A and rotate arms down into driving position. Turn on the device.

WHILL Model A Personal EV meets the following guidelines for safe airline transportation: Federal Aviation Administration (FAA); Transportation Security Administration (TSA); Department of Transportation (DOT); International Air Transport Association (IATA). However, the device has not been specifically tested for airline transportation.